









#### Simple Paediatric & Newborn Screener

The Otocheck LE is an economical handheld OAE hearing screening instrument, designed for simplicity and applications that do not require electronic recording of patient details and results. With 'one press and go' function, a result in seconds, minimal training required, it's ready for any User. Added benefits of low-cost consumables, long battery life and over 30 years of OAE screening expertise, easily make the Otocheck LE the most affordable and fastest solution for OAE screening.



**RAPID**DPOAE Screening



**PAEDIATRIC FRIENDLY**Distraction animated display



FREQUENCY RANGE 2 to 6 kHz Pass SNR 6dB Fixed









## **OVERVIEW**

- Rapid DPOAE screening pass/refer result
- Easy to use 1 key press-and-go operation
- · Minimal training required
- SNR & Signal to noise histogram display
- · Paediatric distraction display with lock safe feature
- · Light-weight and portable with easy-to-use keypad
- Automated self-test system when powering on Otocheck
- Long battery life up to 250 test cycles or one full working week
- · Print wirelessly to the Otodynamics optional mini printer
- Choice of language English, French, Spanish, Portuguese, German, Italian


	DP <b>OAE</b>	
Test Configurability	1 Protocol, 2 Parameters	
Frequency Range	2, 3, 4, 6 kHz	
Freq. Resolution	1/2 octave	
Stimulus Levels	60/50 to 70/70 (6 Presets) (dBSPL)	
Pass Criteria	2, 3, 4 Bands	
Pass SNR	6dB Fixed	





# SIMPLE, EASY DPOAE SCREENING

Minimal training required

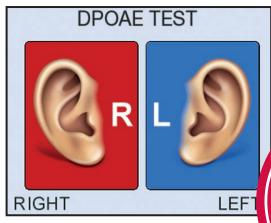

Start a test with one press ear selection. Each test starts with automatic in-ear calibration ensuring accurate results.






### PAEDIATRIC DISPLAY

Highly effective with children, the visual distraction displays provide 6 animations, engaging the child, helping to keep them quiet. The animation gradually fills with colour if the child is quiet, but colour fades when the child is noisy.



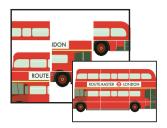






### LOCK SAFE

Engage more with your patient while testing. With one button push, lock the device in paediatric display and allow the patient to hold the Otocheck LE while testing. The patient can navigate the 6 animations while testing without tampering with the instrument functions, which further encourages the patient to be quiet and still through out the test.



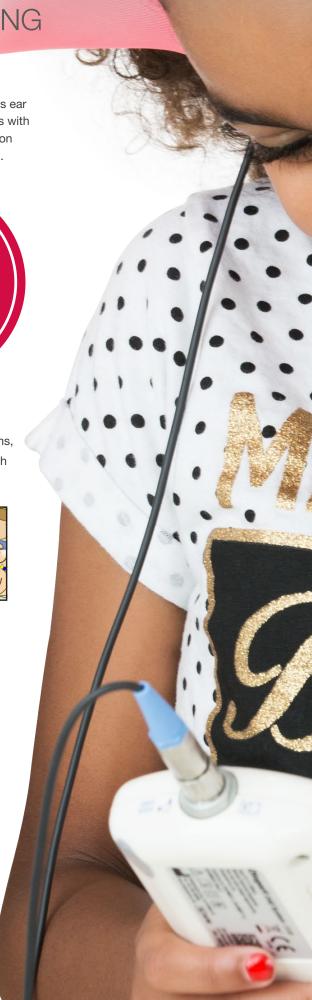









#### Otodynamics Ltd.

30-38 Beaconsfield Road Hatfield Herts AL10 8BB UK τ. +44 (0)1707 267540 ε. info@otodynamics.com www.otodynamics.com