

NEW
2026

OAE/AABR

Otoport^{Pro}



SCREENER
FAST | EASY | CLEAR



CLINICAL
OAEs DESIGNED FOR THE CLINIC



ADVANCED CLINICAL
WITH EXTENDED OAE FUNCTIONS

NEW
2026



Otoport^{Pro}

OAE / AABR

Otoport Pro will meet your needs whatever your clinical requirements, size or type of screening program. It's a compact, easy to operate handheld device featuring unrivalled adaptability.

The Otoport Pro brings together over 35 years of expertise in one portable device. Otodynamics was the first OAE company and we pioneered methods that set the OAE gold standard. The reliability and performance of Otodynamics' AABR screening algorithm has been proven in universal newborn hearing screening programs around the world.

For screening, select the Otoport Pro Screener model with the test combination you prefer. From infant screening with AABR, OAE, or AABR+OAE to Pediatric screening with OAEs. High specificity relies on good testing conditions and here the Otoport Pro Screener scores by assessing the testing environment and providing essential feedback.

For investigations, the Otoport Pro has powerful clinical analysis models with fully customizable OAE protocols. Each test is saved with the actual physiological response. Choose the Otoport Pro Advanced Clinical model for clinical research and access to selected research functions for IRB approved projects.



HIGHLY ADAPTABLE
Meeting your screening
& clinical needs



FULLY CUSTOMIZABLE
Up to 16 protocols
available



ANY TEST COMBINATION
DP | TE | AABR | DP+AABR |
TE+AABR | DP+TE+AABR



VISUAL DISTRACTION DISPLAY
Friendly pediatric animations



QR SCAN
For data entry & login

CHOOSE YOUR OTOPORT PRO MODEL


Every Otoport Pro has the same hardware, ensuring reliability, uniform performance, and upgradability. Three distinct firmware versions provide levels of functionality tailored to three application areas.



The Screener model

SCREENER *Fast | Easy | Clear*

The Otoport Pro Screener has been expertly designed to be flexible, simple, easy-to-use, and fast, while meeting all your screening needs from newborn screening through to pediatric assessments. Screeners are aided to identify problems by on-screen messages, to improve their techniques, and thereby improve overall performance of the program. Choose any one or a combination of DPOAE, TEOAE and AABR tests. Upgrade later with a QR license activation. Each test type delivers reliability and performance with ultra fast, accurate and clear results.

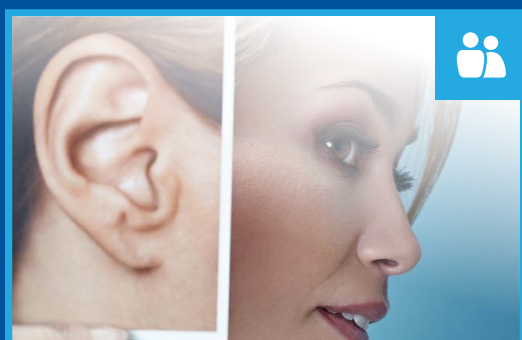

Includes Screener model

CLINICAL *OAEs designed for the clinic*

The Otoport Pro Clinical has exactly what's needed for clinical OAEs at a highly competitive price. Six powerful customizable protocols, including 12 frequency 1-8kHz DPgram with normative range markers. Clinical OAEs have real clinical value with adults and older children, alerting you to the possibility of both conductive and cochlear losses. The Otoport Pro Clinical includes all the Otoport Pro Screener functions and beyond. With one touch, the Otoport Pro Clinical can be configured for adult, pediatric or infant screening applications.



Includes Screener & Clinical model

ADVANCED CLINICAL *With extended OAE functions*

Benefit from the full range and power of the Otoport Pro with the Advanced Clinical. Boasting 16 fully customizable protocols, the Otoport Pro Advanced Clinical model will perform in the most demanding of clinics. Incorporating all the functions of the Otoport Pro Screener and Clinical models, the Advanced clinical has three additional investigational OAE functions. DP ZOOM explores DP-gram features at up to 100 points per octave resolution. Stimulus levels are configurable in 1dB steps. DP GROWTH provides multi stimulus level DPOAE measurements at any frequency 1-8kHz. MULTI TEST function linking up to 4 loop-able test protocols allowing you to build a sequence of different DP and/or TE tests.



FEATURES COMMON TO ALL MODELS

Learn more about the large number of features that are included in every Otoport Pro model.



QUICKLOOK GUIDE

Review the quick look guide to compare. at a glance, each Otoport Pro model and their related DPOAE, TEOAE and AABR specifications. See additional options for your Otoport Pro



Otoport^{Pro} **SCREENER**

FAST | EASY | CLEAR

OAE / AABR

The Otoport Pro Screener comes pre-configured for screening efficiency and arrives ready to go. Whether you choose AABR or OAE as your primary screen, Otoport Pro requires minimal training to achieve fast and clear results. To meet your special requirements, you can also customize test protocols and use OAE and AABR in combination.

With Otoport Pro you get multiple days of use between battery charges. Its robust design, large onboard searchable patient database and detailed response capture, means Otoport Pro can meet all your screening and program management needs.

UNIQUE DISTRACTION DISPLAY


When conducting challenging pediatric OAE assessments, the Otoport Pro has 6 visual distraction animations that helps to keep the child quiet and engaged. The child is encouraged to remain quiet as they watch the onscreen cartoon gradually filling with color. If noise is detected during the test, the colors quickly fade and return when the child settles creating positive reinforcement.

STREAMLINED TESTING

WORK LIST		1/5
ID	FAMILY	
2E3B7605...	Doe	
2E3B7610...	Bloom	
2E3B7628...	Hall	
2E3B7637...	Lewis	
2E3B7647...	Martin	
DETAIL TEST		

Use the programmable Work List feature to plan your screening workload.

PAUSE CONTROL

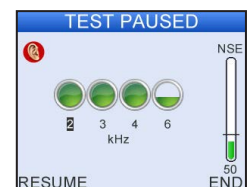

Pause and resume a test at any time to check probe fit, adjust the environment or to allow time to settle a patient.

1. QUICK INPUT



Effortlessly enter **patient** details via QR code scanning*, worklist or enter manually.

2. EASY FIT



Otodynamics right angled probe can be used for both OAE & AABR testing. Its unique design makes it easy to achieve a good probe fit.

Ear cups can also be used with the Otoport Pro. However, using the probe inserted into the ear for AABR can reduce consumable costs. Otoport Pro can perform single or dual probe testing.

3. SUPERIOR PRE-TEST ANALYSIS

IMPEDANCE CHECK	
TN PN	kΩ
+	Forehead 3.3
●	Shoulder 5.0
-	Nape 2.2
	Balance 1.1
BACK CHECK START	



See test progress every step of the way, automated alerts are displayed if changes in stimulus levels, impedance, or excess noise that may effect the test.

For AABRs, superior pre-test environment checks provide separate powerline & infant noise indicators and quickly identify if a specific electrode has high impedance.

Otoport^{Pro} **SCREENER**

CHOOSE YOUR TEST TYPES

DP OAE



✓ **INFANT** ✓ **PEDIATRIC**

Tests 4 frequencies (2,3,4,6kHz) with a choice of 3 preset stimulus levels, optimized for screening. Rapidly test infants and pre-school children.


TE OAE



✓ **INFANT**

Tests up to 5 half octave frequency bands (1,1.5, 2, 3 and 4kHz) using our 'Quickscreen' click stimulation at 84dB SPLpe, proven in the worlds most extensive screening trials. TEOAEs are fast and effective for infant screening.


AABR

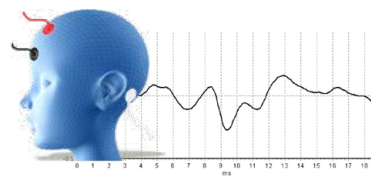

✓ **INFANT**

Tests using short Chirp stimulation, configurable between 35 and 45dBnHL designed for newborn screening. Results are fast and with continuous monitoring of Fsp and ABR waveform shape to ensure that a 'clear response' is a true response.


4. CLEAR TEST PROGRESSION

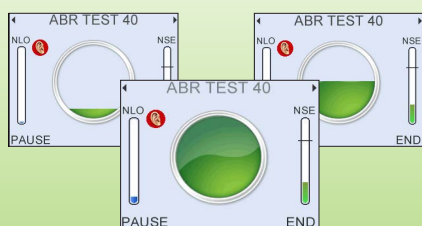

See test progress every step of the way, automated alerts are displayed if changes in stimulus levels, impedances, or excess noise levels that may effect the test.

For AABR there is continuous monitoring of Fsp and ABR waveform shape to ensure that a 'clear response' is a true response.

5. FAST RESULT

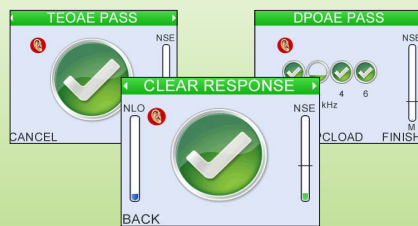

The Otoport Pro Screener is fast, easy and clear. With just 2 key presses**, achieve a clear response.

OAE in as little as 7 seconds
AABR average pass in 80 seconds

6. SMOOTH DOWNLOAD



Data transfer is quick and easy. Simply plug-in and tests can be automatically downloaded to Otolink PC software via wired or wirelessly

* Hospital QR and barcoded IDs vary. The specification of your facilities QR/Barcode format is required to confirm compatibility.

** Quick Start option enabled



Otoport^{Pro} **CLINICAL**

OAEs DESIGNED FOR THE CLINIC

The Otoport Pro Clinical model provides fast and accurate DPOAE recordings on adults, children, infants, and much more with extensive configurability and expandability.

At the heart of the Clinical is a 12 frequency 1-8kHz DPgram with $\frac{1}{4}$ octave resolution with 5 stimulus formats. Smart frequency sweep management optimizes test time and settable normative range markers assist your interpretation of the results.


You can create customized DPOAE test protocols for special requirements. Use multiple levels to record cochlear function to high accuracy e.g. for the detection of changes with therapy or for monitoring cochlear status over time.

Every test stimulus is optimized using our unique 'in-the-ear calibration' process. Test details and responses are saved to the onboard patient records database, which you can search and review on the device. Or transfer results to PC, via USB or Bluetooth download, for more detailed examination with the included Otolink PC software.


The Otoport Pro Clinical includes all the OAE functions of the Otoport Pro Screener with increased protocols and customization.

ADD

AABR

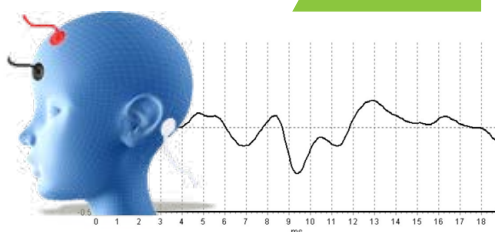


KEY FEATURES

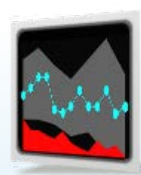

DP 12 FREQUENCY

Two customizable 1-8kHz DPgram protocols with $\frac{1}{4}$ octave resolution and 5 selectable stimulus formats. Smart frequency sweep management optimizes test time and settable normative range markers assist interpretation

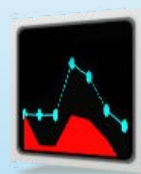

DP OAE

Two 1-8kHz customizable protocols with Half octave linegraph, histogram and frequency circle display.



ADD

TE OAE

Increase the depth and power of your clinical investigations by adding our gold-standard 1-6kHz TEOAE analysis with half octave analysis between 800-6kHz and waveform display

You can add AABR infant screening to the Otoport Pro Clinical model, offering two AABR protocols customizable between 35 - 45dBnHL. Extend to 60dBnHL for non-screening training/research use. Display the emerging ABR waveform test in real time and detailed examination post test.



Otoport^{Pro} **ADVANCED**

WITH EXTENDED OAE FUNCTIONS

Top of the Otoport Pro range, the 'Advanced Clinical' model is an exceptionally powerful handheld auditory investigation device.


The Advanced Clinical has all the features and options of the Otoport Pro Clinical plus three additional investigational test modes: DP Zoom, DP Growth and MULTI-TEST. TEOAE is also included as standard with additional high-resolution spectral analysis from 0.5 to 6kHz.

Take advantage of all Advanced Clinical's sophisticated tools to go beyond routine clinical DPOAE recordings towards clinical research investigations. Capture detailed test information including physiological responses for in-depth analysis in our Otolink PC software, which can also export data for research.

Otoport Pro records high resolution DPOAEs from 0.5kHz to 8kHz. Sweep frequencies automatically or under manual control to focus on difficult frequencies. Achieve unrivalled frequency resolution with DP Zoom. Explore cochlear function with input-output measurements using the Advanced DP Growth test. Link up to 4 tests together in a repeatable test sequence using the MULTI-TEST macro programmer.

ADD

AABR

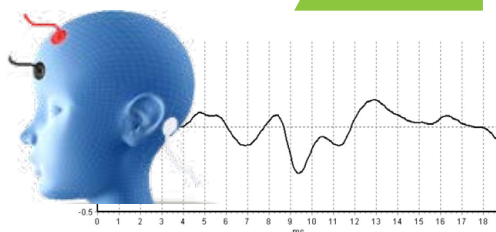

KEY FEATURES




DP **ZOOM**

Allows you to investigate a specific range of frequencies from 0.5 to 8kHz in high resolution of up to 100 pts/oct. Explore details of DP-gram features such as notches to establish their true extent and depth. Export data for analysis and research.



DP **GROWTH**

Select any frequency and see how DPOAE intensity grows with stimulus level. Stimuli are optimized by the 'scissor' formula. Plot either dB/dB or uPa/dB growth. Automatic regression analysis provides slope and DPOAE threshold data for sufficiently linear uPa growth results.

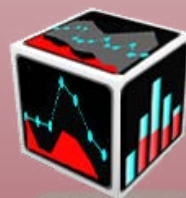

MULTI TEST

Four fully customizable loop-able test protocols. Allows you to build a sequence of OAE and/or AABR tests at different levels. Collect vast amounts of data automatically.

You can add AABR infant screening to the Otoport Pro Advanced Clinical model, offering two AABR protocols customizable between 35 - 45dBnHL. Extend to 60dBnHL for non-screening training/ research use. Display the emerging ABR waveform test in real time and detailed examination post test.

Otoport^{Pro}



COMMON FEATURES ACROSS ALL MODELS

SCREENING STREAMLINED

ID	FAMILY	TEST
2E3B7605...	Doe	
2E3B7610...	Bloom	
2E3B7628...	Hall	
2E3B7637...	Lewis	
2E3B7647...	Martin	

Use the programmable Work List feature to plan your screening workload.

LARGE STORAGE

Thomas	
DPOAE	DPOAE PASS
OAE	OAE1, 70-70dB
OAE	15.7dB
Time	88sec
1.5	2 3 4 6 8
30 Jul 24	10:16 ADN

Store up to 2000* tests listed in chronological order with on-screen test result summary table.

PAUSE TESTING



Pause and resume testing at any time to address any issue impacting a test, such as noise, probe or electrode fitment.

QUALITY ASSURANCE

QA AREA
1 Probe Test
2 OAE Cavity Test
3 Occlusion Test
4 Real Ear Test
5 ABR Cavity Test

BACK ENFORCE SELECT

Dedicated QA test area with optional enforced daily QA. View QA history with on board Probe test facility and Probe ID tracking.

ACCURACY



Every test starts with automated 'in-the-ear' calibration ensuring accurate results.


ADMIN CONTROL

USER	PASSWORD	A
Admin	ADN	A
Doe	DOE	
Evans	EVA	
Hall	HAL	
Dean	DEA	

BACK EDIT

Create multiple user profile logins with password protection and Admin management control and security.

INFORMATIVE

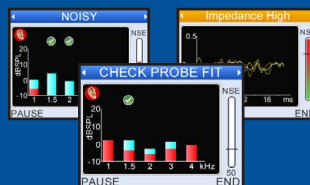

With automated monitoring of the testing environment. Helpful on-screen notification is provided advising of any issue impacting a test such as noise, probe or electrode fitment.

NUMERICAL DISPLAY

L1	L2	SIG	NSE	SNR
70	56.7	69.4	11.5	2.1
65	64.9	64.8	7.4	3.6
60	62.8	59.7	8.3	1.7
55	60.8	54.6	7.2	0.9
50	58.9	49.8	3.0	-0.4
45	57.0	44.4	7.1	-1.4

CANCEL MANUAL END

Numerical table of test data. Signal, noise and SNR.

OTOLINK PC DATA MANAGEMENT SUITE

Download test results wirelessly or via USB from the Otoport or when docked in the docking station. Test details can be quickly reviewed, printed and archived. Printing reports directly to a PDF file is available. Data can be exported, for example, to OZ, HiTrack, GDT, Noah** or as CSV file. Patient Work Lists can be uploaded to the Otoport, assisting with managing workloads in busy environments. Otolink also provides access to firmware and application updates from Otodynamics via the internet. Compatible with Windows 10 and 11.



* dependent on Otoport Pro device.

** additional option

UNIQUE DISTRACTION DISPLAY

When conducting challenging pediatric OAE assessments, the Otoport's has 6 visual distraction displays that help to keep the child quiet and compliant. The child is encouraged to remain quiet as they watch the onscreen cartoon gradually fill with color. If noise is detected during the test, the colors quickly fade and return when the child settles, engaging the child by positive reinforcement.



SCANNING


Built in QR/Barcode reader. Effortlessly scan patient details for easy data input or to search existing patient records**.

Scan user details for login access.

To upgrade Otoport Pro or to tailor the test mode for specific applications, simply scan a different QR code license***. With the Otoport Pro you are not limited to one installation.




Transfer data to PC via Bluetooth® or wired connection.

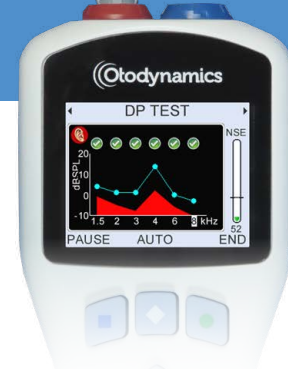

Easy push pull probe connection. Choice Single or dual probe testing


CONNECTIONS

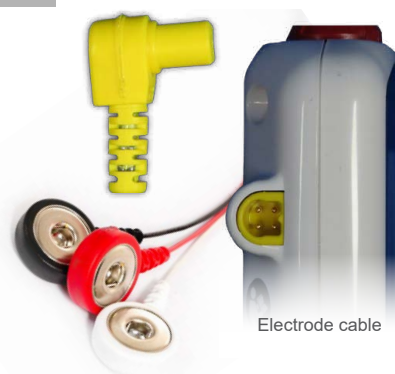

Electrode cable

PROBE & TIPS


The Otodynamics XPD probe is designed to make daily maintenance and cleaning simple and quick. The new TIP HOLDER is fitted over the probe body and locates precisely with the sound ports when the TIP HOLDER tabs are clicked securely on the probe body.

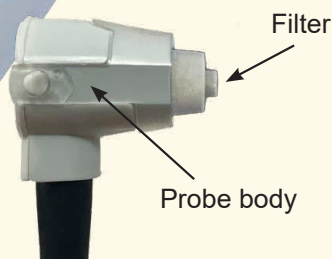

Screening - Tip holder



General - Tip holder

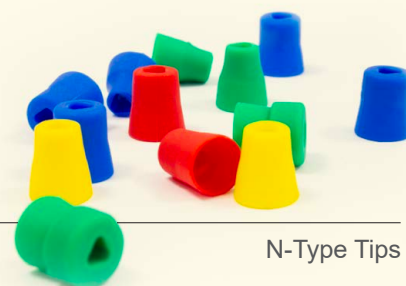

N-Type Tips




R-Type Tips

The XPD probe body benefits from the inclusion of an internal filter that fully protects the microphone and loudspeaker ports.

** Hospital QR and barcoded IDs vary. The specification of your facilities QR/Barcode format is required to confirm compatibility.
*** additional option.

ADDITIONAL OPTIONS

for your Otoport Pro

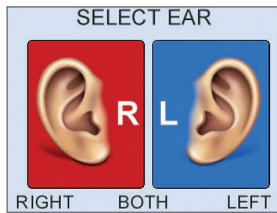

ADD BINAURAL TEST

Seamless testing both ears reduces patient disturbance.



ADD MINI PRINTER

Wireless, compact, fast print time, numerical and band pass data. Battery powered.



Noah 4


ADD NOAH

To the Otolink PC Data Management.



ADD CARRY CASE

To store and transport the Otoport Pro and all related items.

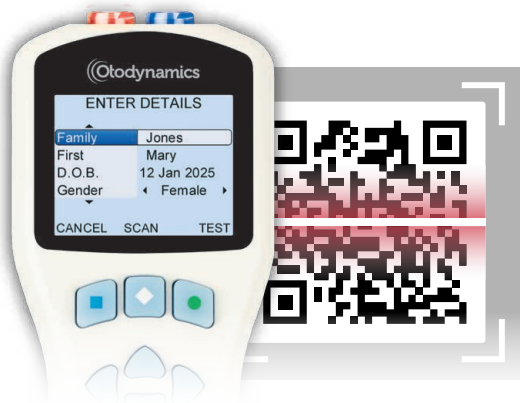

ADD SCANNER

Built in QR/Barcode reader. Effortlessly scan patient details for easy data input or to search existing patient records**.

Scan user details for login access. To upgrade Otoport Pro or to tailor the test mode for specific applications, simply scan a different QR code license***. With the Otoport Pro you are not limited to one installation.

** subject to advance information of facility's QR specification
*** charges apply



HANDHELD

Otoport^{Pro}

WHAT'S INCLUDED

Otoport^{Pro}



- OAE probe with cable clip
- Probe test cavity
- USB download cable
- Docking station
- Sampled ear tips /infection control sleeve
- Otolink PC software (download)
- User manuals (download)
- Electrode Cables (with AABR option)

QUICK LOOK Otoport^{Pro}

NEW



SCREENER



CLINICAL



ADVANCED CLINICAL

APPLICATION PACKS

NEWBORN SCREENING
PEDIATRIC SCREENING
CLINICAL AUDIOLOGY
RESEARCH

	DPOAE	DPOAE	DPOAE
TEST CONFIGURABILITY	11 Parameters, 1 Protocol	20 Parameters, 4 Protocols	24 Parameters, 8 Protocols
PRESET FREQUENCY (kHz)	2, 3, 4, 6 kHz, 4 frequencies	Up to 8 kHz, 13 frequencies	Up to 8kHz, 16 frequencies (audiometric or extract values)
F2 FREQUENCY RANGE (kHz)	2 - 6 kHz	1 - 8 kHz	0.75 to 8 kHz
FREQUENCY RESOLUTION	Half octave	Up to 4 pts/octave	Up to 100 pts/octave with DP Zoom
FREQUENCY RATIO	1.22 (Fixed)	1.22 (Fixed)	1.22 (Fixed)
STIMULUS LEVELS L1/L2 (dB SPL)	60/55 or 65/50 or 65/55	60/50 to 70/70 (6 Presets)	60/50 to 70/70 (6 Presets) Custom 40 to 75 (Δ1dB)
DISPLAY	Graphic progress indicators / Numeric / Pediatric distraction / Line Graph	Graphic progress indicators / Numeric / Pediatric distraction / Line Graph / Histogram	Graphic progress indicators / Numeric / Pediatric distraction / Line Graph / Histogram
PASS CRITERIA	2, 3 or 4 frequencies	2 to 6 frequencies, 12 frequency DP	1 to 6 DP or 12 freq DP or 10-100% freq. with DP zoom
PASS SNR	6dB (Fixed)	6dB Configurable up to 14dB (Δ1dB) for automation. OFF for clinical evaluation. 3 noise criteria levels	6dB Configurable up to 14dB (Δ1dB) for automation. OFF for clinical evaluation. 3 noise criteria levels
CPT CODE	92587, 92558	92587, 92558, 92588	92587, 92558, 92588
FEATURES	Rapid DPOAE screening mode Interactive Pediatric distraction display Easy-to-use pass/refer result Automatic test progression and interpretation	Extensive configurability including 12 frequency DPOAE Configurable norms Includes all SCREENER functions Easy switching between applications	DP Zoom to 100pts/octave DP Growth I/O with scissor mode Access to selected research function for IRB approved projects. Multi test macro (4x protocols) Includes all CLINICAL functions Easy switching between applications
	ADD TEOAE or TEOAE only	ADD TEOAE	TEOAE INCLUDED
TEST CONFIGURABILITY	14 Parameters, 1 Protocol	25 Parameters, 2 Protocols	28 Parameters, 2 Protocols
FREQUENCY RANGE (kHz)	1, 1.5, 2, 3, 4 kHz	1, 1.5, 2, 3, 4, 6 kHz	1, 1.5, 2, 3, 4, 6 kHz
FREQUENCY RESOLUTION	Half octave bandwidth	Half octave bandwidth	Half octave bandwidth
STIMULUS LEVELS L1/L2 (dB SPL _{pe})	Fixed 84 dB SPL _{pe}	60 to 90 dB SPL _{pe} (Δ1dB)	60 to 90 dB SPL _{pe} (Δ1dB)
DISPLAY	Graphic progress indicators / Numeric / Pediatric distraction / Histogram	Graphic progress indicators / Numeric / Pediatric Distraction / Histogram / TE Waveform	Graphic progress indicators / Numeric / Pediatric Distraction / Histogram / TE Waveform / high res TE spectrum / Stimulus Waveform & Spectrum
PASS CRITERIA	Customizable SNR in any of 2, 3, or 4 half octave bands, or wide / narrow bands	Customizable SNR in any of 1 to 5 half octave bands	Customizable SNR in any of 1 to 5 half octave bands
PASS SNR (dB)	6dB (Fixed)	6dB recommended variable up to 14dB (Δ1dB)	6dB recommended variable up to 14dB (Δ1dB)
CPT CODE	92587, 92558	92587, 92558	92587, 92558
FEATURES	Gold standard TEOAE Quickscreen Interactive Pediatric distraction display Easy-to-use pass/refer result Automatic test progress and interpretation	Up to 6kHz with configurable norms Choice of response window Easy access to all Otoport Pro SCREENER features with additional protocol.	Access to Otoport Pro CLINICAL features
	ADD AABR or AABR only	ADD AABR	ADD AABR
TEST CONFIGURABILITY	8 Parameters, 1 Protocol	11 Parameters, 2 Protocols	16 Parameters, 2 Protocols
STIMULUS LEVELS (dB nHL)	30 to 45 (Δ1dB) (up to 40 with earcups)	30 to 45 (Δ1dB) (up to 40 with earcups)**	30 to 45 (Δ1dB) (up to 40 with earcups)**
STIMULUS TYPE	Chirp	Chirp	Chirp and Click
CPT CODE	92650	92650	92650
FEATURES	Superior pretest environment, with power-line & individual impedance analysis. Fast pass/refer result Choice in stim delivery ear cup/insert	Real time ABR waveform view ** 45 to 60 dB nHL insert only for training Includes all SCREENER functions. Easy switching between applications with additional protocol.	Real time ABR waveform view **45 to 60 dB nHL insert only for training Extendable FSP for AABR waveform capture Easy access to all Otoport Pro CLINICAL features
ADDITIONAL OPTIONS			
Wireless Module	INCLUDED	INCLUDED	INCLUDED
Docking Station	INCLUDED	INCLUDED	INCLUDED
Integral Barcode / QR Scanner	OPTIONAL	OPTIONAL	OPTIONAL
Add Binaural Testing	OPTIONAL	OPTIONAL	OPTIONAL
Wireless Printer	OPTIONAL	OPTIONAL	OPTIONAL
Add Carry Case	OPTIONAL	OPTIONAL	OPTIONAL



Otodynamics Ltd.

30-38 Beaconsfield Road Hatfield Herts AL10 8BB UK

T. 1-800-659-7776 E. info@otodynamics.com

www.otodynamics.com